


## QUICK REFERENCE FOR CARDIA DELIVERY SYSTEM – May 2011

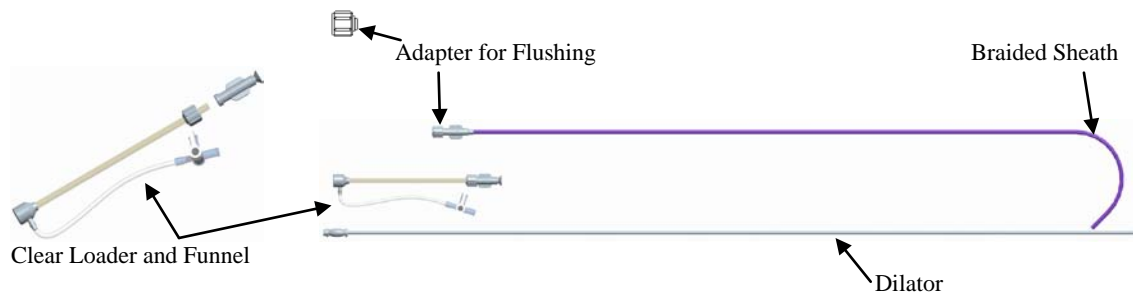

### The Cardia Delivery System contains:

1. 80 cm Braided Sheath with adapter for flushing
2. Dilator
3. Clear Loader and loading funnel

Step 1: The adapter on the Braided Sheath is to flush the long sheath with a syringe. Once the long sheath has been flushed remove the adapter and save.

Step 2: Push the dilator fully into the Braided Sheath

Step 3: Braided Sheath and Dilator are then moved up the Super-Stiff Wire as with any Septal Closure Procedure

Step 4: Once Sheath is in place remove the Super-Stiff Wire and Dilator. Next use the Adapter with a Syringe to “plug” the Braided Sheath.

Step 5: Load ultrasept device using Loading Funnel. As always during device loading, flush the system through the Side-Port with a syringe. Once device is loaded remove Loading Funnel.

Step 6: Remove Adapter and Syringe on Braided Sheath allowing blood to flow. While flushing the Clear Loader insert it into Braided Sheath and twist the lock to secure. Continue to flush device while advancing device through the Braided Sheath

